ETA629790000

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ETA SMART

History

- 2 October 2024
 - 131/91 (81) Blood Pressure 09:45
- 1 October 2024
 - 121/80 (60) Blood Pressure 08:58
- 30 September 2024
 - 130/97 (86) Blood Pressure 12:24
 - 126/96 (87) Blood Pressure 12:23
 - 124/88 (92) Blood Pressure

Values of the last measurement, including date of measurement

Graphical representation of measurement results for the last month

Graphical display of results for the last 7 days

Ability to filter results by measurement type (not required)

List of individual measurements

Pairing and managing devices

Blood Pressure
ETA Blood

Last measurement
131/91 (81)
2 Oct 2024 09:45

Last 30 days

- 7% Optimal
- 32% Normal
- 26% High Normal
- 35% Grade 1 Hypertensi...
- 0% Grade 2 Hypertensi...
- 0% Grade 3 Hypertensi...

Last 7 days

mmHg

| | | | | |
|-----|-----|-----|-----|-----|
| 150 | 145 | 127 | 121 | 131 |
| 108 | 97 | 92 | 80 | 91 |

Home History Devices Export Menu

Manual input of results

Select Health Record

- ETA Blood My Health Record
- AN Abraham Nawal male
- JC Jaroslav Chudý male
- TB Tomas B. male, age 37

New Health Record

Switching and profile list (multiple profiles can be used with a subscription)

- Account
- Health Records
- Shared Access
- Data Sync
- Notifications
- Settings
- Help
- About
- Bonuses & More

Try Health Diary by MedM
One app for logging 20+ measurement types same account & subscription

- Account management, logout, subscription information
- Sharing results with others
- Data transfer to selected third-party applications
- Manage notifications of new measurements or limit value violations
- Units of measurement, change in methodology of results, PIN lock of the app, setting sounds and setting the colour theme of the app
- External websites with information
- Basic info about the application
- Switches to the web portal and displays information on MEDM services

MEDM promotional offer (other app)

Blood Pressure

Time Interval: Last 30 days

03/09/2024 - 02/10/2024

Format: Generic CSV

Action: Share

Export

Export

- All Data
- Blood Pressure
- Weight

Export of measurement results
(The application offers the possibility to export measurements from various devices (not only the pressure gauge) that are paired with the account. When you select blood pressure, the export options are displayed.)

The application allows you to display results in different ways - individually or collectively using graphs.

Blood Pressure
ETA Blood

Last measurement
131/91 (81)
2 Oct 2024 09:45

Last 30 days

- 7% Optimal
- 32% Normal
- 26% High Normal
- 35% Grade 1 Hypertensi...
- 0% Grade 2 Hypertensi...
- 0% Grade 3 Hypertensi...

Last 7 days

mmHg

Home History Devices Export Menu

List of individual measurements

History

Tap to connect

2 October 2024

09:45 131/91 (81)

1 October 2024

08:58 121/80

30 September 2024

12:24 130/97 (86)

12:23 126/96 (87)

12:09 124/88 (92)

12:07 124/89 (94)

11:05 127/91 (81)

10:58 130/88

Each measurement can be individually opened and then deleted, or its parameters can be edited.

Details

1 Oct 2024 08:58

121 / 80 60

Body position: Sitting

Measured arm: Left

Irregular pulse: No

Scale: ESH/ESC (Europe)

Source: ETA6297

Feeling: Good

Note:

You can change the values, time, or circumstances of the measurement, including the ability to add a note.

Edit Measurement

121 / 80 60

08:58 Yesterday

Feeling: Good

Body position: Sitting

Measured arm: Left

Irregular pulse: Not Detected

Note:

Save

Basic graphical representation

3 Sep - 3 Oct

35%

- 7% Optimal
- 32% Normal
- 26% High Normal
- 35% Grade 1 Hypertension
- 0% Grade 2 Hypertension
- 0% Grade 3 Hypertension

Measurement dispersion

3 Sep - 3 Oct

- 7% Optimal
- 32% Normal
- 26% High Normal
- 35% Grade 1 Hypertension
- 0% Grade 2 Hypertension
- 0% Grade 3 Hypertension

Blood pressure and pulse charts

Sys/Dia Chart

25 Sept 2024 - 2 Oct 2024

Day Week Month Year

It is possible to switch between a graph for blood pressure and heart rate

Charts

Sys/Dia Chart

Pulse Chart

Pulse Chart

25 Sept 2024 - 2 Oct 2024

Day Week Month Year

Graphs can be displayed for different time periods

BASIC INFORMATION

The MEDM BP app is part of an ecosystem of smart health applications that connect various measuring devices, allowing for effective monitoring of different body parameters.

This ecosystem also includes a web portal that extends the app's functionality:

(<https://health.medm.com>).

MEDM provides basic services for **FREE**, but also offers enhanced features via a **PAID** subscription.

Basic features available for free:

- Automatic data collection from an unlimited number of connected devices.
- Manual entry of measurement data.
- Use of the app with or without registration.
- Online data backups for registered users.
- Measurement history, trends, and graphs.
- Basic data export.

Premium (paid) features:

- More **health profiles** for family members (in the basic mode, you can only have 1 profile).
- **Data synchronization** with connected health ecosystems (Apple, Garmin, Google, Fitbit, etc.).
- **Health record sharing** (you can share your account and activity with others, such as grandparents with children, who can help monitor your health).
- **Remote monitoring** (via the app or the MedM Health Portal).
- **Notifications** – alerts for threshold measurement values, reminders, and goals (you can set PUSH notifications to your phone or email for each new measurement, or only for measurements exceeding tracked threshold values).
- and more...

Important notes

- These basic features are sufficient for regular use and monitoring of blood pressure. Premium features only expand the monitoring and control of measurement results – **they are not mandatory and do not need to be paid for.**
- Premium features (subscription) are offered and provided by MEDM and are subject to their terms. These are not services provided by ETA, so we do not guarantee their quality, stability, or any issues that may arise from their use.
- Subscription prices are listed on the MEDM website or directly in the app.
- ETA has negotiated a special offer for its customers – after connecting and pairing the ETA6297 blood pressure monitor, the app will automatically offer a 50% discount on the subscription. If you are interested in premium services, pair the monitor first to apply the discount.
- A detailed overview of the premium features can be found at the end of this manual.

eta

Enabling Connected Health®

for Service Providers, Device Vendors, Developers, People

DeviceKit SDK

Self & Family Care

White-label RPM

Soft HomeHub

USER MANUAL FOR THE MEDM BP APPLICATION

Dear customer,

Thank you for your trust in ETA products. This manual is intended for all owners of the ETA629790000 device. Here, you will find simple guides and instructions on how to use your new assistant.

Additional information, advice, and tips for your device can also be found on our website:

www.etasince1943.com/care


We constantly update these websites, gathering the latest information about the product.

Note

We reserve the right to make changes to the app, its function, and appearance. Since the app is a dynamic element of the product, some procedures or illustrations mentioned in this manual may differ.

1. INSTALLING THE APP

You can find the app in the Apple Store or Google Play app databases, or you can use the following QR code for quicker access: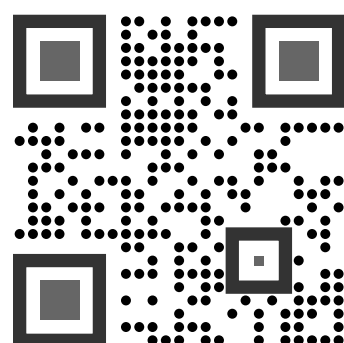

Download and install the app on your mobile phone (or tablet).

COMPATIBILITY


| | |
|---|---|
| Mobile phone (tablet) operating system: | Android 8.0 or higher IOS 14.2 or higher |
|---|---|

Usage Tips

- The app allows various ways to export and display measured results. You can further work with this data, e.g., on a PC. However, if you want to share only a specific graph (e.g., a monthly blood pressure curve), create a screenshot of your mobile device and then share this image via email or social networks as you usually do. Note – each type of mobile phone may have a different method of creating these screenshots. If you don't know how, check your phone's user manual or search online.
- In the app, you can track long-term trends in blood pressure values. We recommend regularly analyzing the results and consulting them with your doctor if necessary.
- The language of the app is based on your phone's system settings. However, in some cases, to apply the change, you may need to clear the app's cache in your phone's settings for the specific app, or reinstall the app.

2. ACCOUNT CREATION AND LOGIN

1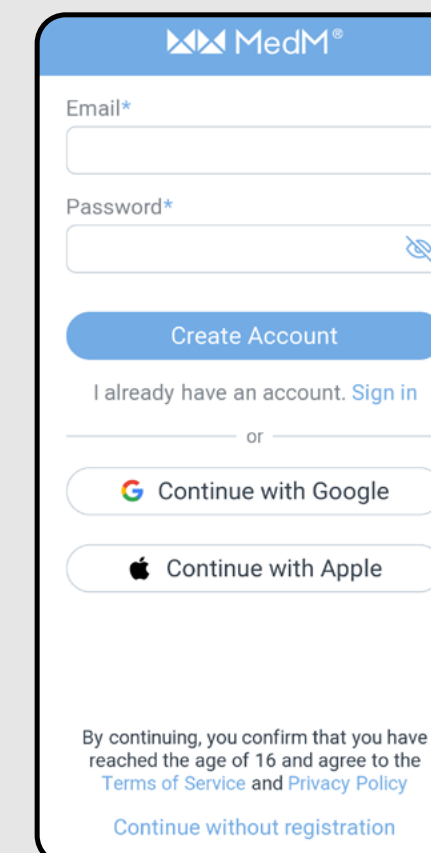


- Open the MEDM BP application.
- If you already own an ETA device and are using it with the MEDM BP app, proceed to "**Sign in**," where you will enter your email and password, and you will be automatically logged in.

Note:

A guest mode ("**Continue without registration**") is also available. This mode is mainly intended for testing the app with limited functionality. For regular use, we recommend creating an account.

2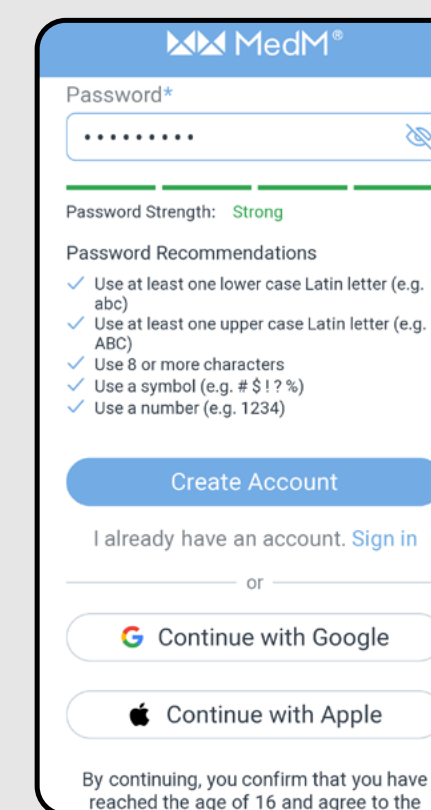


- To register, enter the email you want to use for your account.
- Then, create a strong password.
- Click on „**Create Account**“.

Note:





The app also offers the option to create an account and log in using Google or Apple accounts. If you wish to use this option, click on "**Continue with Google/Apple**".

3
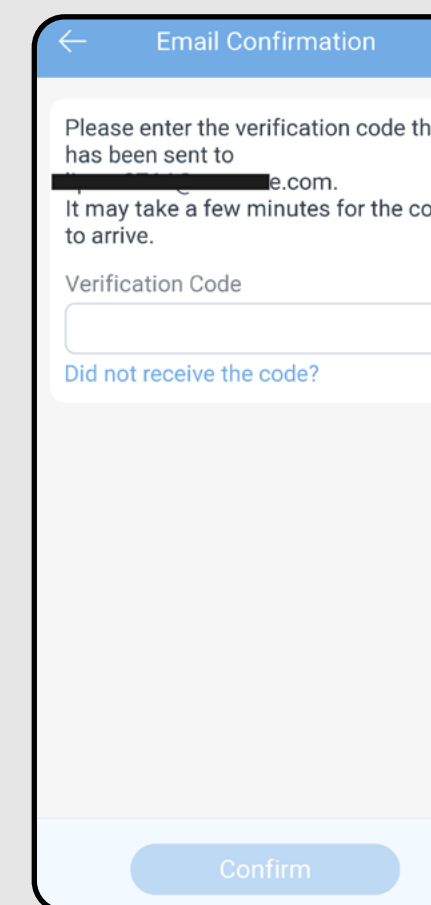





- An individual verification code will be sent to your email.
- Enter the verification code.
- Continue by clicking „**Confirm**“.

ETA629790000

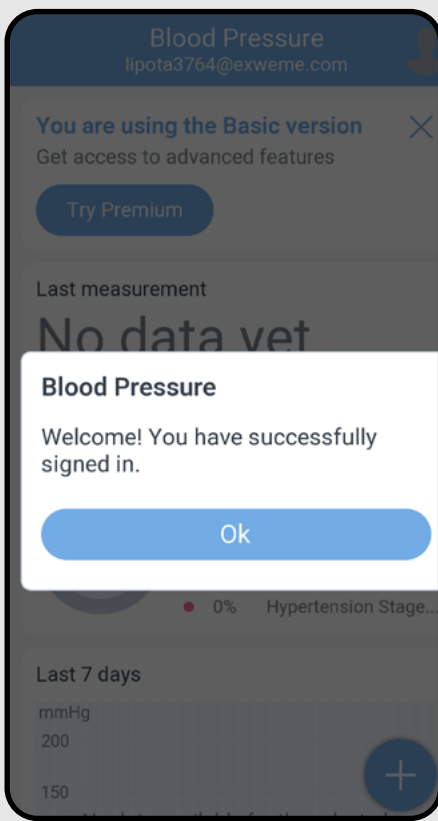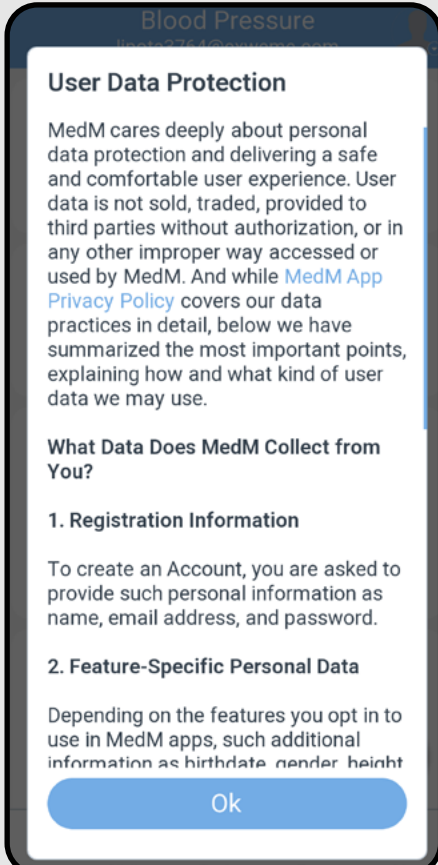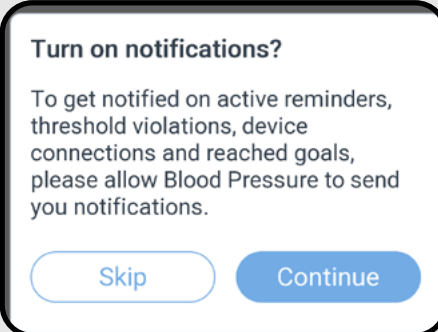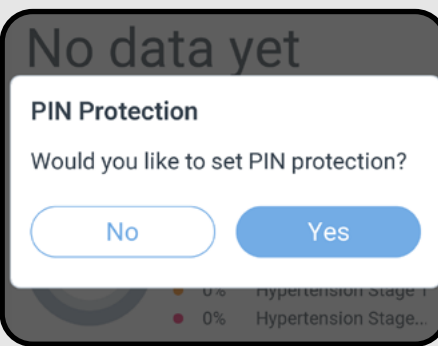
| | | |
|---|--|---|
| 4 |  | <ul style="list-style-type: none"> Depending on the type of phone, you may be automatically logged in and directly enter the app. Confirm by pressing "OK". |
| 5 |  | <ul style="list-style-type: none"> Depending on the type of phone and operating system, you may be asked to agree to the MEDM app's terms of use. Confirm by pressing "OK". <p style="text-align: center;">Note:</p> <ul style="list-style-type: none"> If prompted in further steps, allow the app all necessary permissions. |
| 6 |   | <ul style="list-style-type: none"> In the following steps, the app will ask if you want to activate notifications (🔔) and secure access to your account with a PIN code. For the basic free version, click "Skip" on notifications. |

3. DEVICE PAIRING
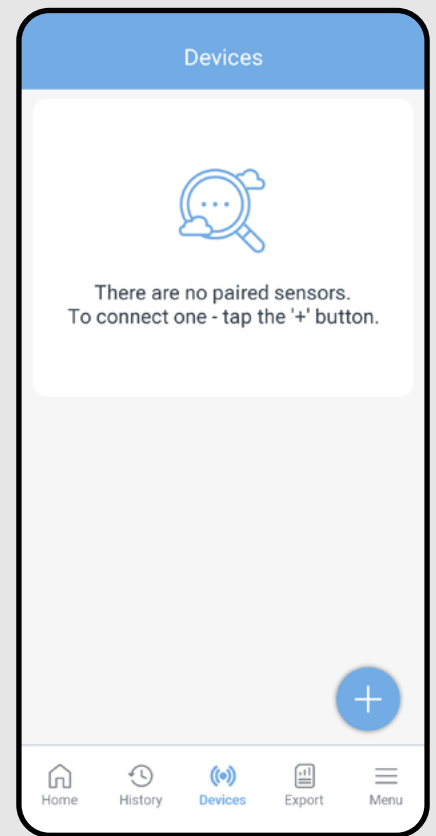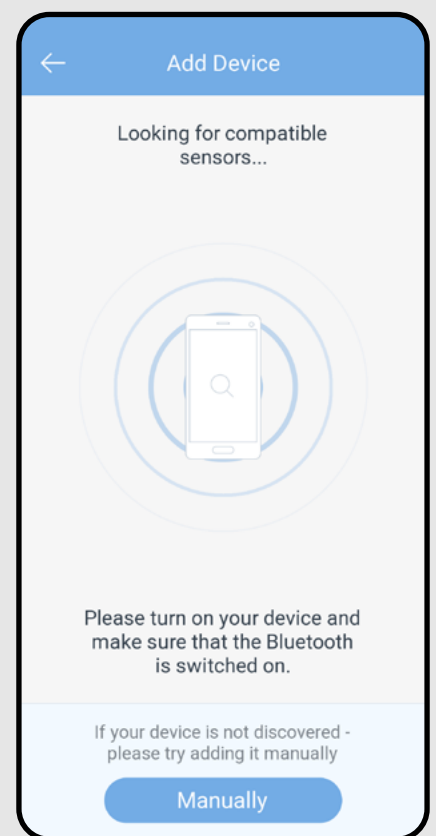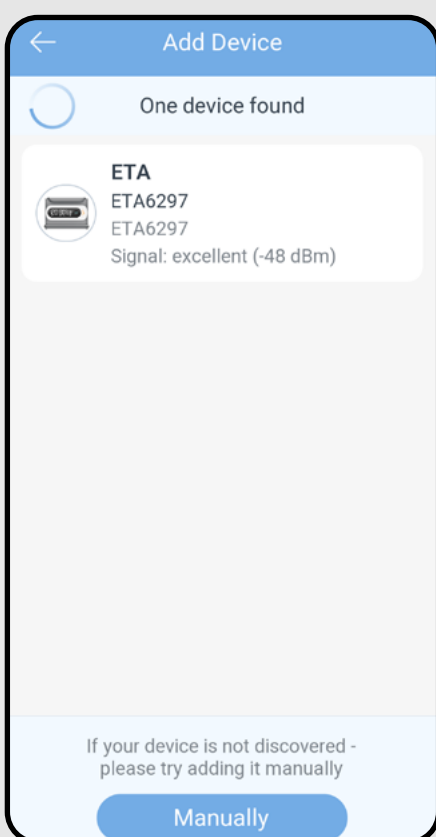
| | | |
|---|---|--|
| 1 |  | <ul style="list-style-type: none"> First, ensure that the blood pressure monitor's battery is sufficiently charged. Open the MEDM BP application and log in to your account. Then click on „Devices“. To add a new device, click on "+". |
| 2 |  | <ul style="list-style-type: none"> The blood pressure monitor can be paired in two different ways: 1) Automatically - the application will find activated devices nearby. 2) Manually - from the product list, manually find the ETA 6297 device and select it. |
| 3 |  | <p style="text-align: center;">1) Automatic Pairing</p> <ul style="list-style-type: none"> During the search for compatible devices (see the previous screenshot), press and hold the main button on the blood pressure monitor for 5 seconds until the Bluetooth icon starts flashing. The monitor will start transmitting a signal, and the app will automatically find and display it. After confirmation, click on the found device ETA6297. |

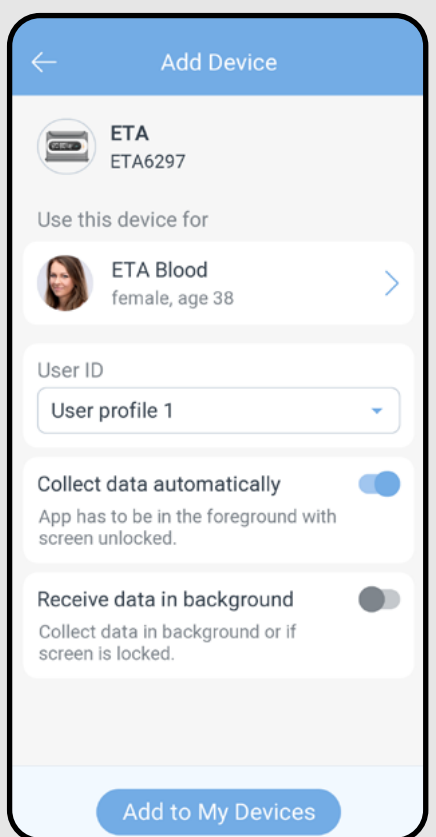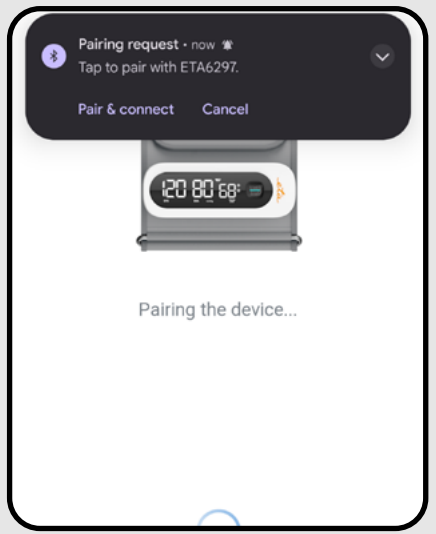
| | | |
|---|---|---|
| 4 |  | <ul style="list-style-type: none"> Here, select which profile will own this device and receive its results (if you have a paid premium and multiple profiles). You can also select User Profile 1 or 2, which correspond to U1 and U2 on the blood pressure monitor. The "Collect data automatically" option allows automatic transfer after the measurement is completed. If this option is not active, you will need to request the measurement results manually. This is related to the "Receive data in background", option, which you should select if you want to receive data even when the app is closed. Confirm the addition by clicking on "Add to My Devices". |
| 5 |  | <ul style="list-style-type: none"> Depending on the type of phone and operating system, the application may display a request for pairing confirmation at this step. Confirm by clicking on "Pair" (or similar). |
| 6 |  | <ul style="list-style-type: none"> The application will begin the pairing process. At this point, the Bluetooth symbol on the blood pressure monitor must still be flashing. |

7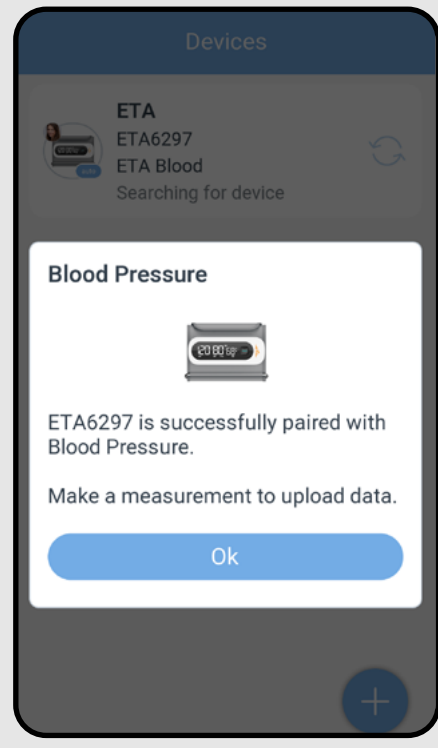

- Once paired successfully, confirm by clicking "OK".


8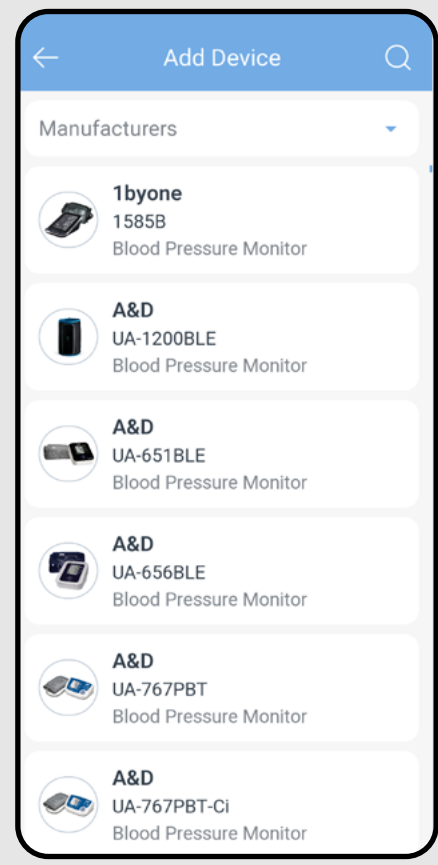

2) Manual Pairing

- Click on „**Devices**“.
- Once the app starts searching for nearby devices, click on „**Manually**“.
- A list of devices from different product categories and manufacturers will appear.
- Click on the "🔍" icon.

9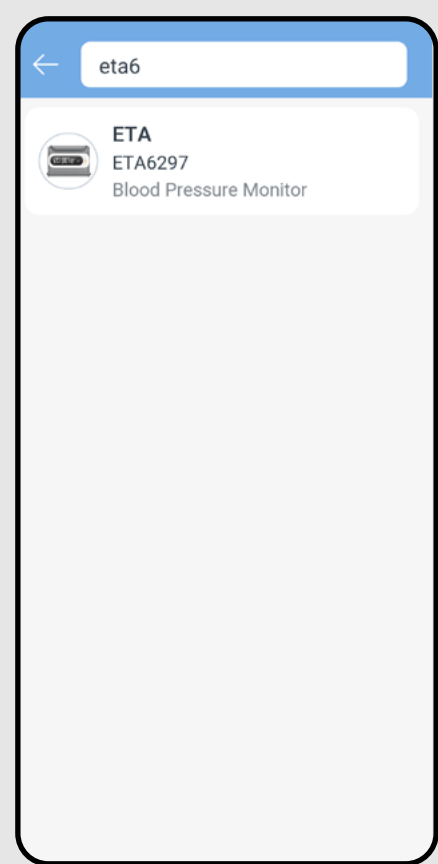

- Then enter the code designation of the blood pressure monitor ETA6297.
- The monitor will appear here. Click on it and proceed the same way as with automatic pairing (the process moves to step 3.4)
- Again, you will need to press and hold the main button on the blood pressure monitor for 5 seconds until the Bluetooth icon starts flashing.

NOTES:

- If the app does not display the blood pressure monitor in the list of found devices during automatic pairing, try restarting the app and starting the pairing process again. Alternatively, restart the pairing process on the blood pressure monitor itself (press the control button for 5 seconds).
- Only use smart devices with supported Android and iOS versions.
- Keep your smart device close to the blood pressure monitor (within approximately 2 meters) during pairing.
- Try pairing with another phone to rule out issues with the phone.


5. DEVICE REMOVAL

1

- Click on „**Devices**“.
- Then click on the paired device field.


2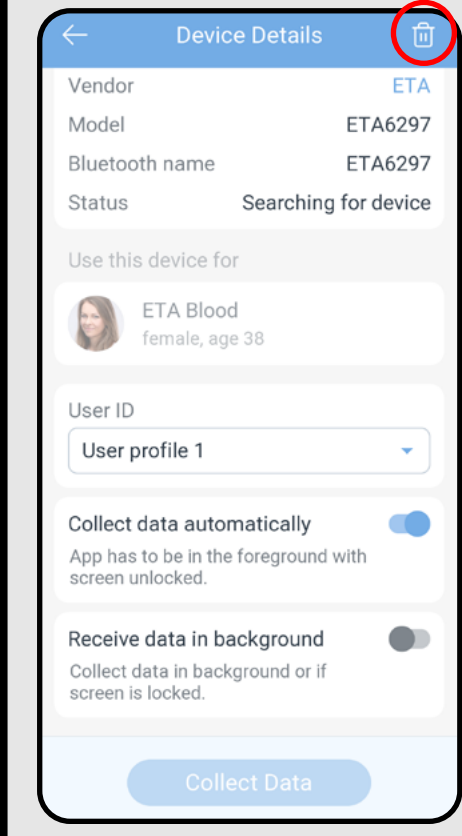

- The **Device** card will open.
- To delete (or unpair) the device, click on the icon (🗑️) and then confirm.

6. MEASUREMENT AND DATA TRANSFER

1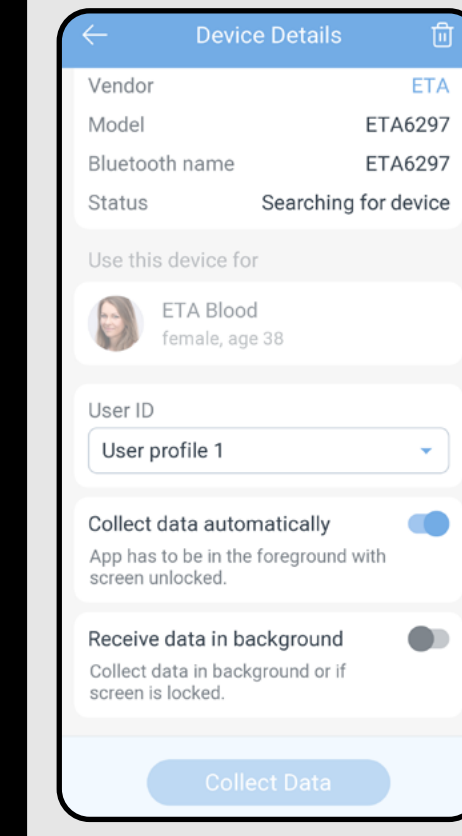


- Depending on the settings of the device (see step 3.4), results can be received: **A) Automatically** after each measurement (if the phone is on, nearby, and has Bluetooth active), or **B) By manually** transferring data.
- For automatic data collection, keep the "**Collect data automatically**" option activated – we recommend using this setting.

Note

If, for some reason (e.g., Bluetooth is off on the phone), the data is not transferred, it will be transferred during the next measurement.


2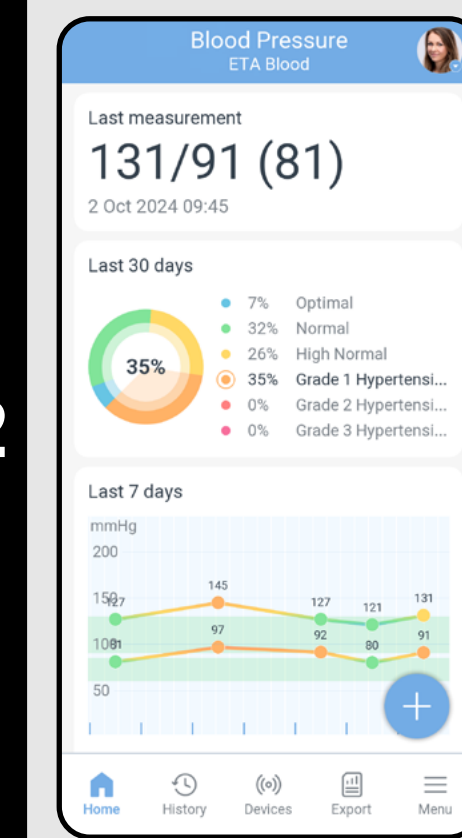

A) Automatic Data Transfer

- Before the measurement, make sure the MEDM application is running for automatic data collection.
- According to the instructions included with the blood pressure monitor, turn on the charged monitor and select the user for whom you want to transfer the measurement results to the app.
- After a successful measurement, the blood pressure monitor will transmit its data for a while, and if the MEDM app is active at the same time, the results will automatically transfer to the selected user's profile.

3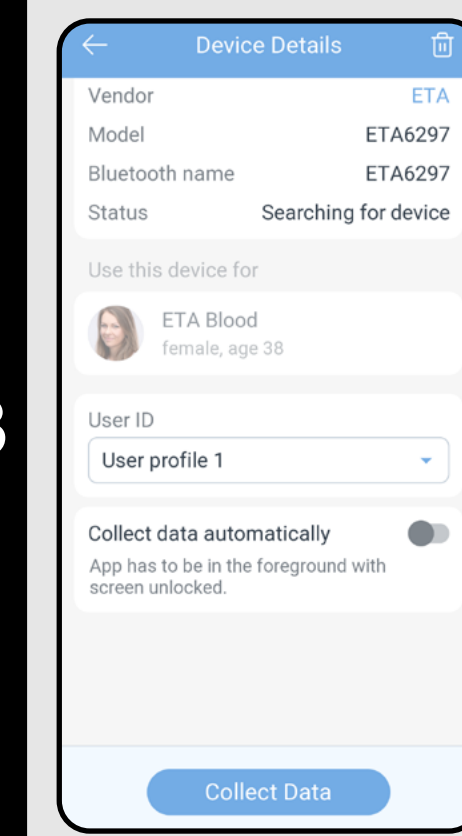


B) Manual Data Transfer

- First, measure your blood pressure the same way (see above) – it can be measured repeatedly, as the monitor has internal memory.
- When you want to transfer the results from the monitor to the app, click on "**Devices**" in the bottom panel of the app's home screen.
- Then, the monitor will need to be set to data transmission mode – this can be done either during the blood pressure measurement or by pressing and holding the button for 5 seconds.
- Click on the paired device field and then click on "**Collect Data**".

ETA629790000

4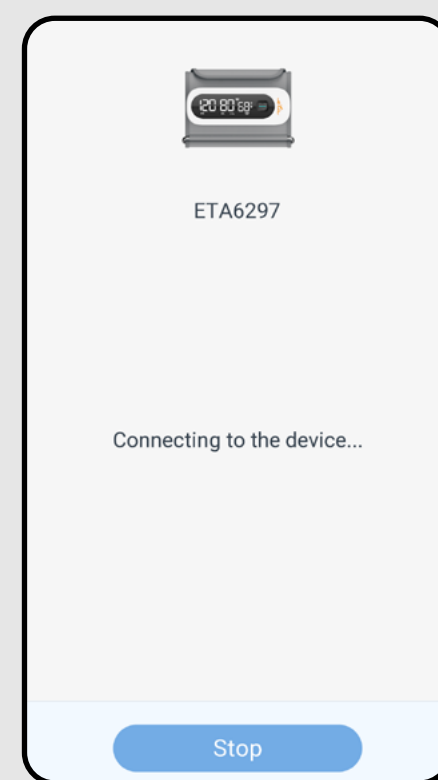


ETA6297

Connecting to the device...

Stop

- The app will start searching for the paired blood pressure monitor.

5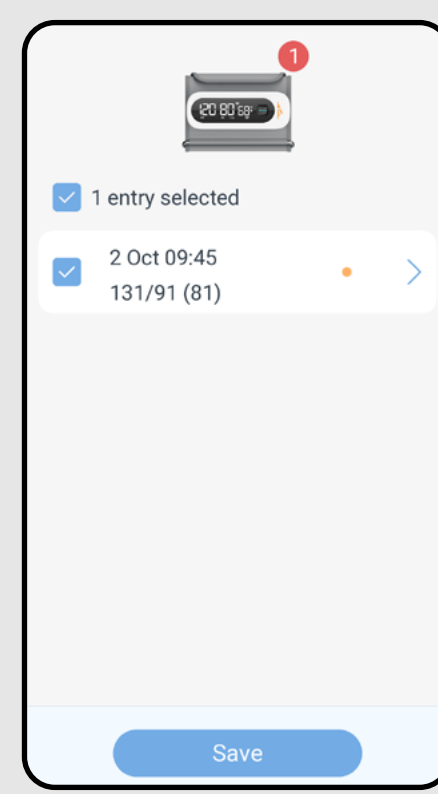


1 entry selected

2 Oct 09:45
131/91 (81)

Save

- Once the data is found, all measurements will be displayed, and the app will offer you the option to transfer them.
- To save the records, click on "**Save**".

2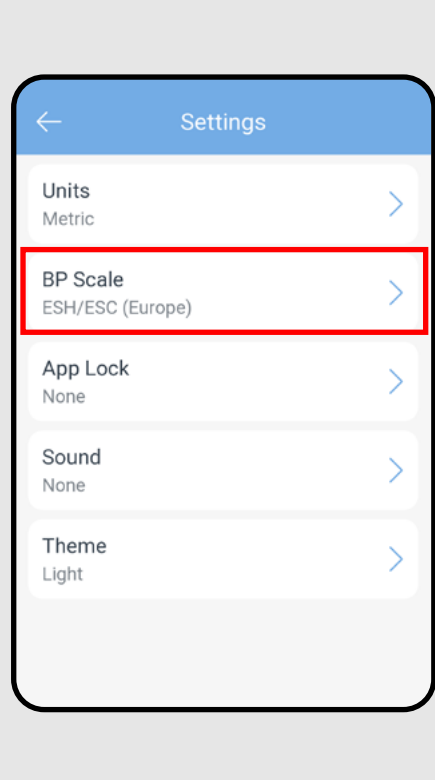

Settings

Units Metric

BP Scale
ESH/ESC (Europe)


App Lock None

Sound None

Theme Light

- Click on „**BP Scale**“.

3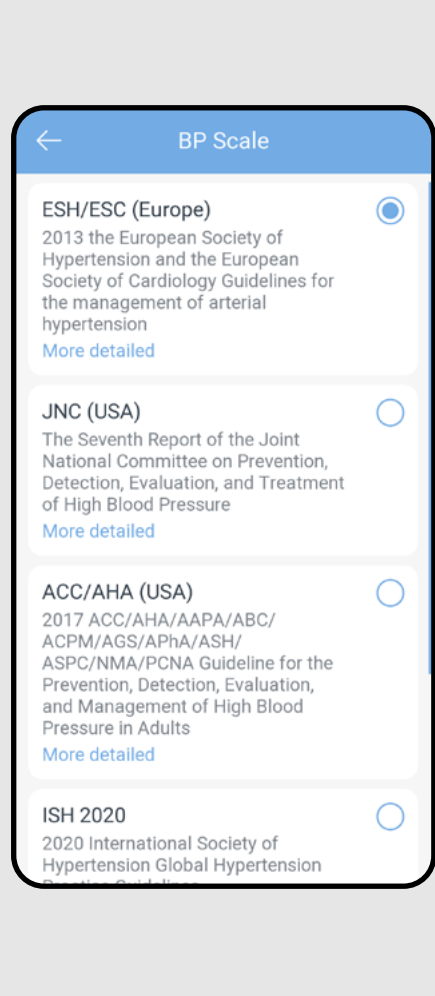

BP Scale

ESH/ESC (Europe)
2013 the European Society of Hypertension and the European Society of Cardiology Guidelines for the management of arterial hypertension
More detailed

JNC (USA)
The Seventh Report of the Joint National Committee on Prevention, Detection, Evaluation, and Treatment of High Blood Pressure
More detailed

ACC/AHA (USA)
2017 ACC/AHA/AAPA/ABC/ACPM/AGS/APHA/ASH/ASPC/NMA/PCNA Guideline for the Prevention, Detection, Evaluation, and Management of High Blood Pressure in Adults
More detailed

ISH 2020
2020 International Society of Hypertension Global Hypertension


- Here, you can change the measurement classification. The choice of classification affects how the results are displayed and evaluated.
- If you're unsure about this, select the European classification, which is standard in Europe.

TROUBLESHOOTING

For troubleshooting related to the app, visit the help page: www.etasince1943.com/care.

8. PREMIUM SERVICES - ACTIVATION

1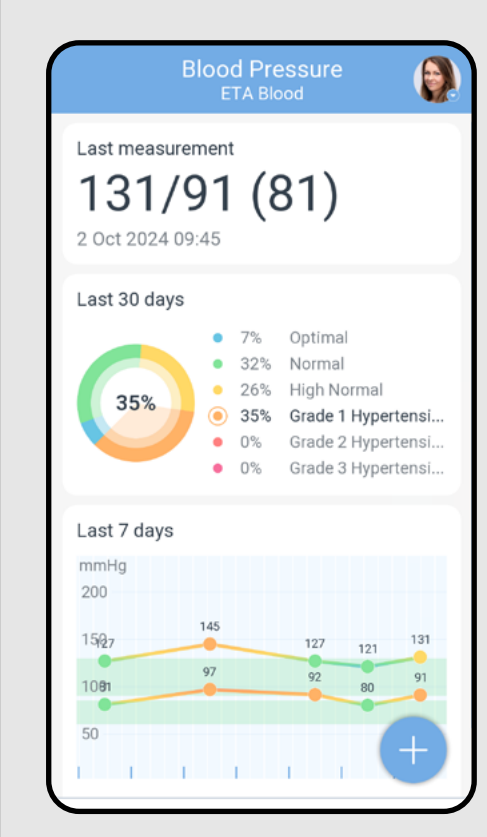

Blood Pressure
ETA Blood


Last measurement
131/91 (81)
2 Oct 2024 09:45

Last 30 days
35%
7% Optimal
32% Normal
26% High Normal
35% Grade 1 Hypertensi...
0% Grade 2 Hypertensi...
0% Grade 3 Hypertensi...

Last 7 days
mmHg
200
150
100
50

- After successful pairing, the app will automatically display a 50% discount offer for ETA customers.
- If you do not want to use the premium services, close this offer by clicking the "**X**" in the top right corner of this message.
- If you want to take advantage of the offer, click on „**Claim Special offer**“.

2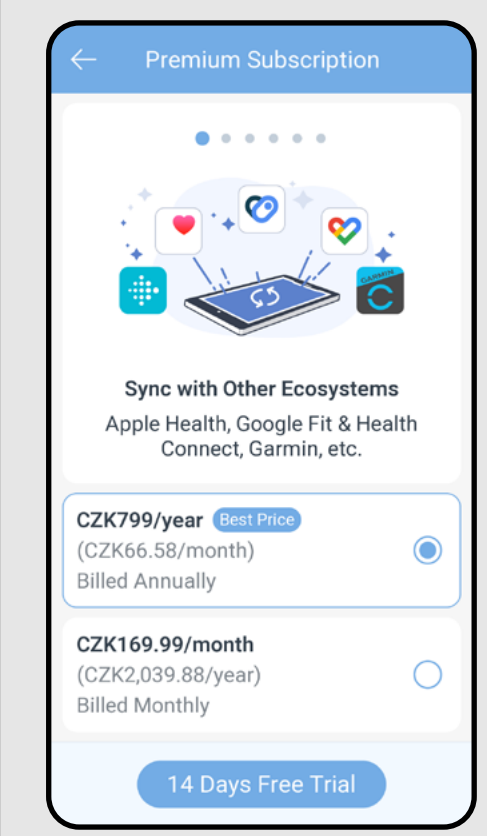

Premium Subscription

Sync with Other Ecosystems
Apple Health, Google Fit & Health Connect, Garmin, etc.

CZK799/year **Best Price**
(CZK66.58/month)
Billed Annually


CZK169.99/month
(CZK2,039.88/year)
Billed Monthly

14 Days Free Trial

- From the options, choose your preferred subscription duration (annual is usually more cost-effective).
- Confirm by clicking "**14 Days Free Trial**".

7. MEASUREMENT CLASSIFICATION

1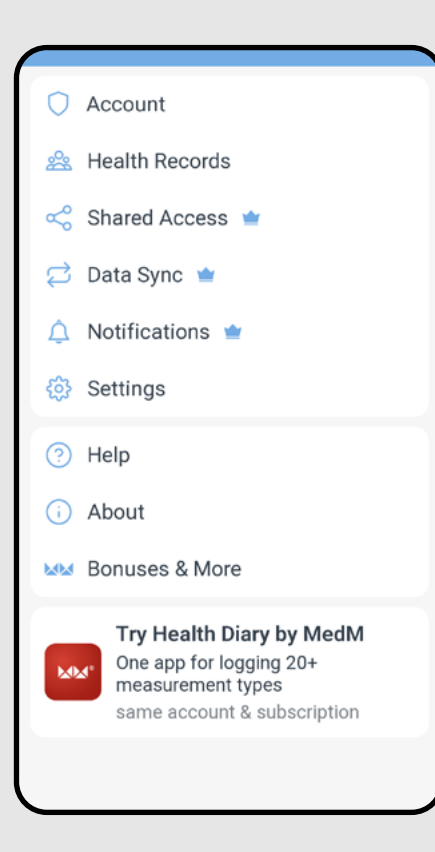

Account

Health Records

Shared Access

Data Sync

Notifications

Settings

Help


About

Bonuses & More

Try Health Diary by MedM
One app for logging 20+ measurement types same account & subscription

- Click on „**Menu**“ in the bottom panel of the app.
- Then go to „**Settings**“ by clicking on it.

3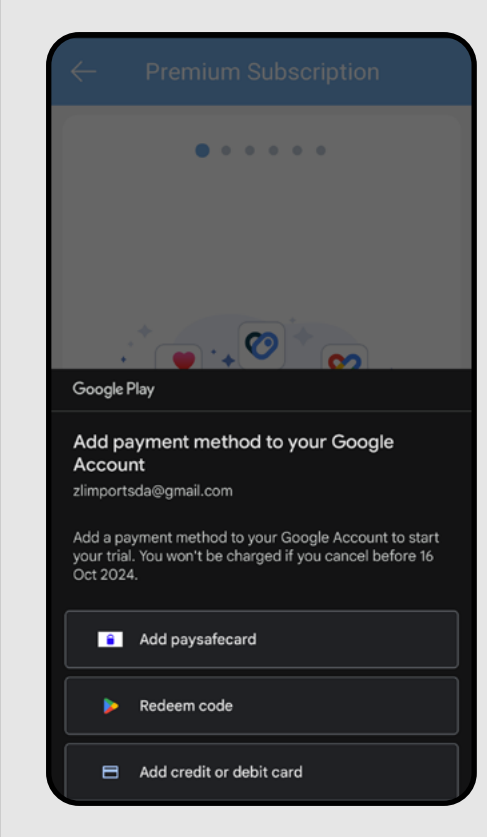

Premium Subscription

Google Play

Add payment method to your Google Account
zlimportsda@gmail.com

Add a payment method to your Google Account to start your trial. You won't be charged if you cancel before 16 Oct 2024.

Add paysafecard


Redeem code

Add credit or debit card

- Then, select your preferred payment method and complete the process.

- If you prefer to use a computer and pay online, you can purchase the premium services directly on the web portal.
- Log in with the same account (email) that you use in the MEDM app.
- Then click on "**Try Premium**", choose your preferred subscription duration, and proceed to the payment methods by confirming "**Buy now**".


<https://health.medm.com>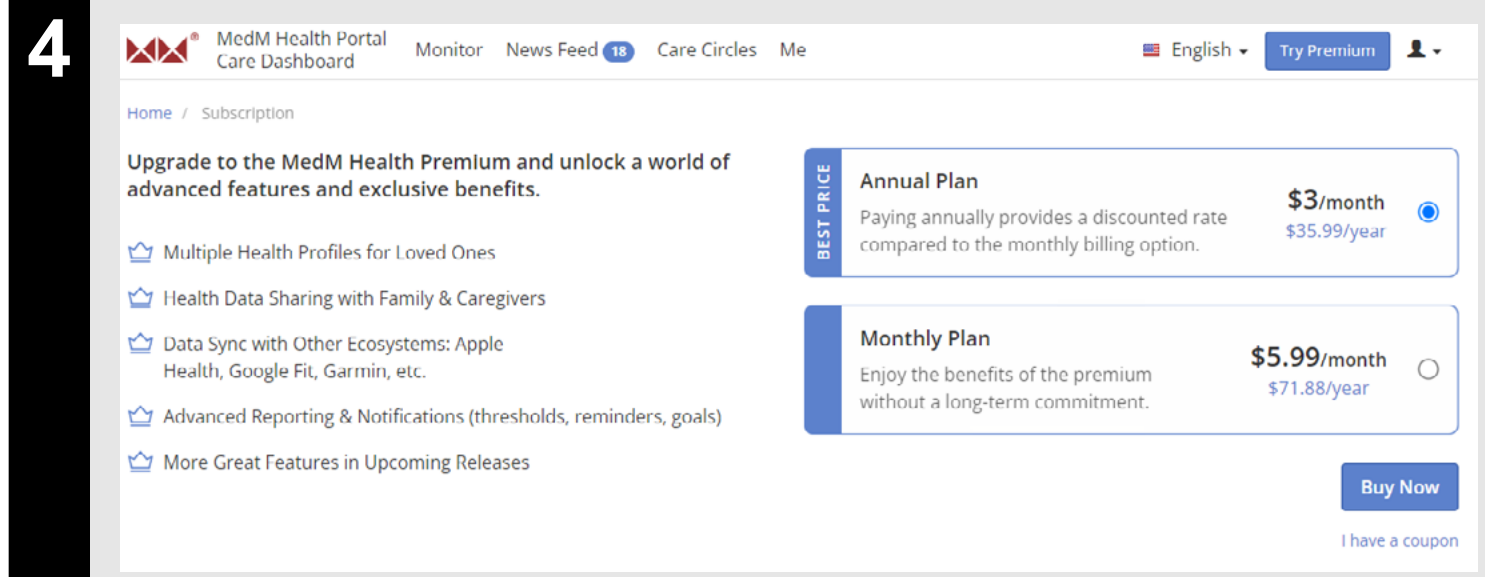

- NOTES:**
- The activation process for premium services, including payments, is not provided by ETA. Any inquiries or potential complaints regarding the subscription should be directed to MEDM (support@medm.com).


9. PREMIUM SERVICES - SHARING ACCESS

1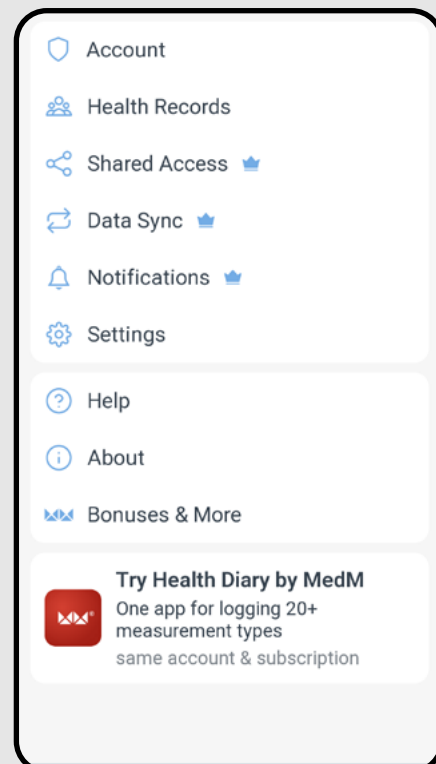


- Click on „**Menu**“ in the bottom panel of the app.
- Then go to „**Shared Access**“ by clicking on it.

2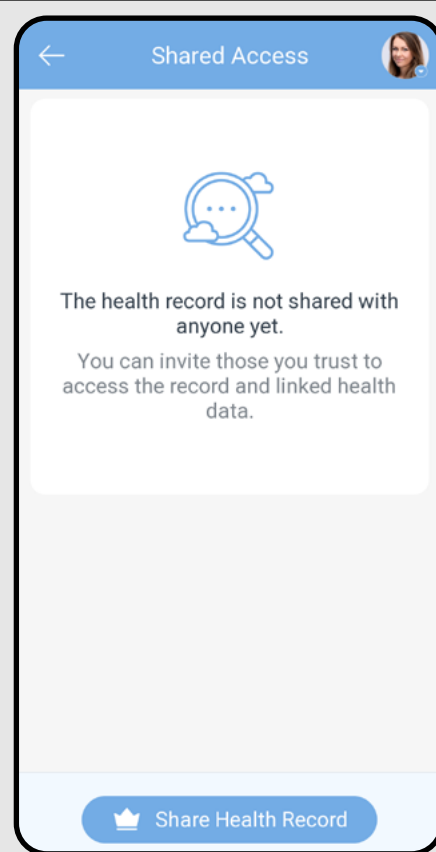


- Here, you can create connections where you provide your measurement results to other people with a MEDM account.
- This is not about sharing specific results but the entire account – giving a live view into your measurement history.
- To create sharing, click on "**Share Health Record**".

3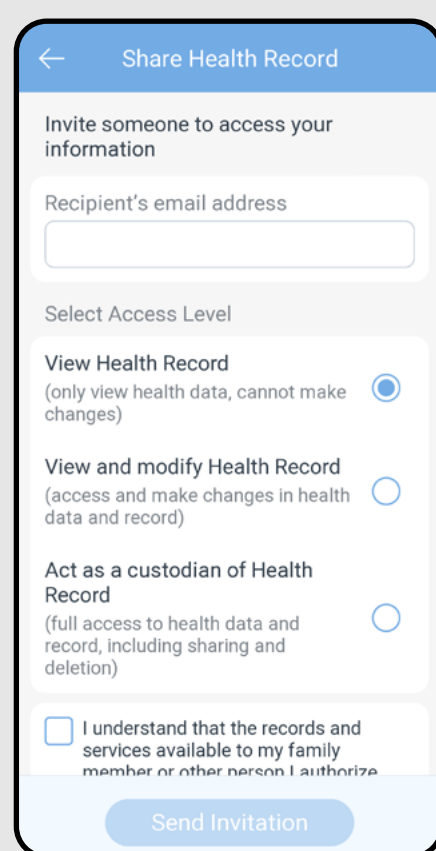


- At the top, enter the email address of the person you want to grant access to.
- In the next section, choose the level of access – whether you allow your results to be only viewed, edited, or full access where the person can perform other settings, including sharing with other people.
- Finally, confirm by clicking on "**I understand that...**" and send the invitation by clicking on "**Send Invitation**".

4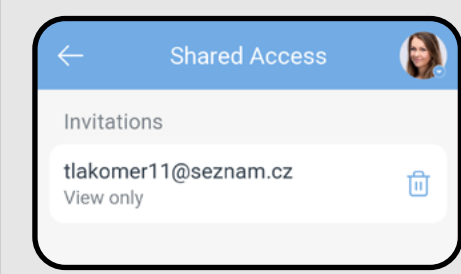


- Now, an invitation will be sent to that person's email to create a profile in MEDM and either accept or reject your request (located in the "**Care Circles**" section in the web portal interface, see step 5 below).
- The app screen shows pending invitations. At this stage, they can still be revoked by clicking on the (🗑️) icon.

5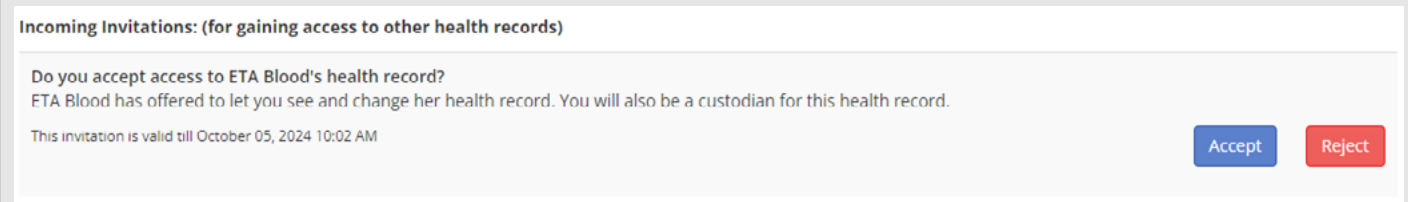

- Once the person accepts the invitation, the option to revoke it will disappear from the app.
- Depending on the permissions you granted to the person, you can cancel the sharing.
- In the case of full access, only the person who was granted full access can cancel this sharing from the web portal.


6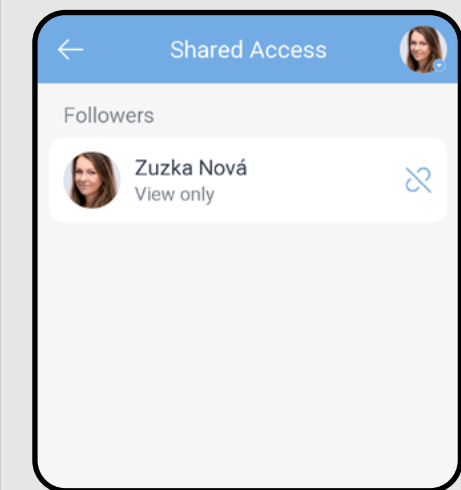

- NOTES:**
- The person you granted access to will see your profile in their MEDM app.
 - The person you granted access to does not need to have a paid subscription – however, the person who wants to share their records must have a paid subscription.
 - The web portal provides a more comprehensive management of shared records, including the ability to set and view threshold values.

10. PREMIUM SERVICES - NOTIFICATIONS

1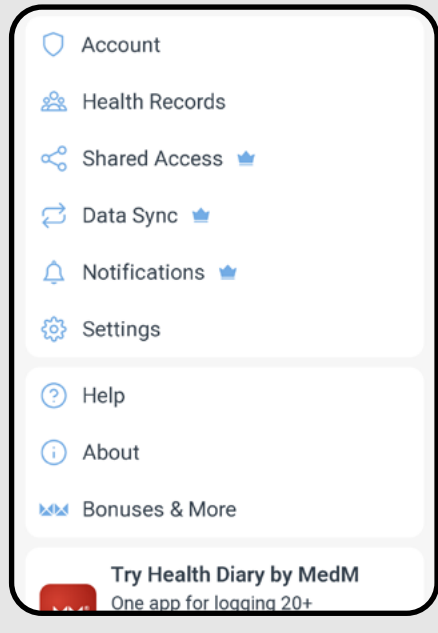


- Click on „**Menu**“ in the bottom panel of the app.
- Then go to „**Notifications**“ by clicking on it.

NOTES:

- Email notifications may be in English.
- PUSH notifications work depending on the permissions and settings of your phone.

2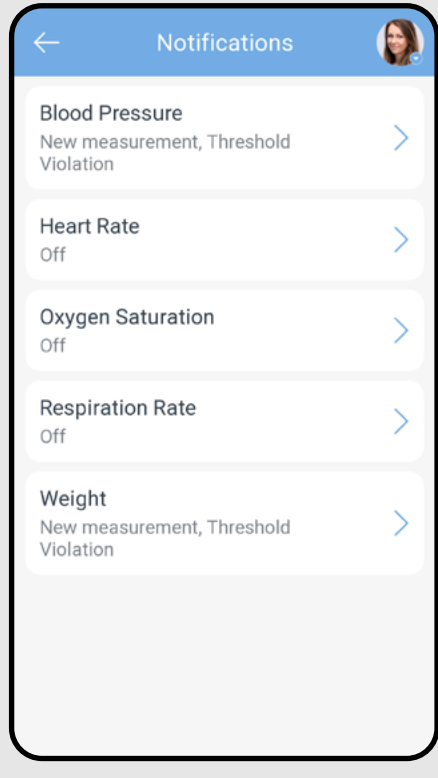


- Depending on which devices are paired with the account and what values are measured, you will see several options for tracked values. In this case, select "**Blood Pressure**".

WEB PORTAL

(<https://health.medm.com>)

The services of the web portal expand the possibilities of working with measurement results. However, for regular use and full functionality, they make more sense if you are using a paid subscription.

3

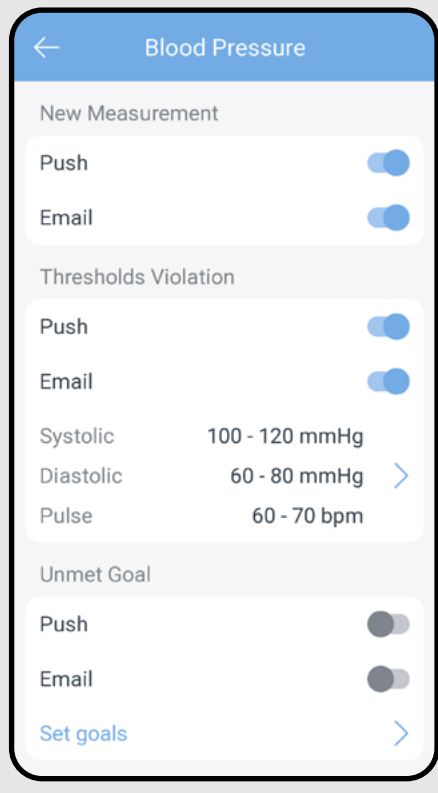- In the "**New Measurement**" section, you can activate the sending of PUSH messages (standard notifications on the mobile phone) or Email messages for each new measurement.
- In the "**Threshold Violation**" section, you can set threshold values for blood pressure and pulse that you consider normal for your measurements. Here too, you can activate PUSH and Email notifications.
- In the "**Unmet goal**" section, you can set notifications for missed measurement goals. An example of a goal could be measuring once a day and seven times a week. This function helps monitor regular blood pressure measurements.

HU

ETA629790000

ETA SMART

History

- 2 October 2024
 - 131/91 (81) Blood Pressure 09:45
- 1 October 2024
 - 121/80 (60) Blood Pressure 08:58
- 30 September 2024
 - 130/97 (86) Blood Pressure 12:24
 - 126/96 (87) Blood Pressure 12:23
 - 124/88 (92) Blood Pressure

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